

# Molecular Collection Device/Test Guide

## Cytology Primary Collection Devices



### ThinPrep Pap

**Test (Stability):**  
 Pap (90 Days)  
 HPV (14 days)  
 HPV Genotyping (14 Days )  
 CT/NG (30 Days)  
 Trichomonas (14 Days)



### SurePath Pap

**Tests (Stability):**  
 Pap (28 Days)  
 HPV (7 Days)  
 HPV Genotyping (7 Days)  
 CT/NG (7 Days)



### Sterile Specimen Cup

**Test (Stability):**  
 Urine for Cytology (48 Hours)  
 CT/NG (24 Hours)  
 M gen (24 Hours)  
 Trichomonas (24 Hours)

## Molecular Primary Collection Devices



### Aptima® Multitest Swab Kit

**Test (Stability):**  
 CT/NG (60 Days)  
 Trichomonas (60 Days)  
 Candida Vaginitis/Trichomonas (30 Days)  
 Bacterial Vaginosis (30 Days)  
 HSV 1&2 (36 Days)  
 M gen (60 Days)




### Aptima® Urine Collection Kit

**Test (Stability):**  
 CT/NG (30 Days)  
 HSV 1&2 (36 Days)  
 M gen (60 Days)  
 Trichomonas (30 Days)



### Aptima® Unisex Collection Kit

**Test (Stability):**  
 CT/NG (60 Days)  
 M gen (60 Days)  
 Trichomonas (60 Days)

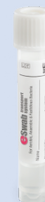

### eSwab (Liquid Amies)

**Test (Stability):**  
 GBS (24 Hour)